GELITA-SPON® STANDARD

Resorberbar gelatinesvamp-hämostatikum (USP)

1. Produktbeskrivelse

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Absorbable Gelatin Sponge Hemostat (USP)

1. Product Description

The GELITA-SPON® STANDARD Absorbable Gelatin Sponge Hemostat (USP) is made of 100% purified gelatin. It is intended for use as topical hemostat. It is insoluble in water and fully biodegradable. Its porous structure and inertness enables it to absorb up to 40 times its own weight of blood and fluids. The absorbent capacity of GELITA-SPON® STANDARD is a function of its physical size, increasing as the amount of materials increases. In addition, it has the ability to release drugs. It may be used as a carrier for any drug (the product literature for the applicable drug should be consulted for complete prescription information). It is supplied in several sizes. It is packaged individually in (double) blister or container packaging, sterilized by means of Gamma irradiation, and is for single use only.

2. Indications

Topical hemostat use as an adjunct to hemostatic tamponade effect, in particular where control of capillary, venous, and arterial bleeding, by pressure, ligature, and conventional procedures, is either ineffective or impractical.

In 21 Dental surgery, the GELITA-SPON® STANDARD Cube is an aid in providing hemostasis and filling dead space created by extraction of the teeth, root amputations and removal of cysts, tumors and impacted teeth.

2.2 For ENT surgery "High Density Sponges" have been shown to be effective in supporting and keeping the fascia or perichondrium in the middle ear dry during surgery in place of the outer ear canal after tympanoplasty.

2.3 For total surgical hemorhoidectomy and gynecology the GELITA-SPON® STANDARD "Tampon" form is available.

3. Contraindications

GELITA-SPON® STANDARD should not be used in cases of skin incisions as it may interfere with healing of the skin edges. This is due to mechanical interposition of gelatin and not to intrinsic interference with wound healing.

The product should not be used in antibiotic infected wounds.

4. Side effects

There have been no reported adverse reactions for product used correctly according to these Instructions for Use and when the product is not over-packed.

Formation of tissue granulation during middle ear procedures has been reported in at least one animal study.

5. Directions for use

Sterile technique should be applied when using GELITA-SPON® STANDARD. The gelatin sponge can be used dry or saturated with a sterile physiological saline solution.

If used, the sponge is cut into the desired size and is slightly compressed. The sponge must be applied to a bleeding area under light pressure for one or two minutes and the bleeding stops.

When used with saline, GELITA-SPON® STANDARD should be soaked in the solution, withdrawn, squeezed thoroughly to expel air bubbles present in the interstices, replaced in saline, and kept there until needed.

When bleeding is controlled, the material can be left in place; however excess should be removed.

Depending on the site of use and the type of surgery, GELITA-SPON® STANDARD is totally biodegraded in less than 4 weeks. When applied to bleeding areas, it should be removed within 2 to 5 days.

5.1 ENT sponge: The gelatin sponge is cut to fit the otic, nasal or oral cavity or defect and inserted to support and separate tissues and to control bleeding if tamponade effect. The excess should be removed!

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Ne pas re-stériliser.

6. Reactions to the package

In cases where the gelatin sponge has been compressed (either by the manufacturer or end-user), the gelatin sponge expands with the absorption of fluid (the sponge re-expands to its original shape). In radical cavities; laryngectomy procedures; around or in proximity to forming bone; areas of bony confine; the spinal cord; the optic nerve and chiasm; or close spaces with bone, GELITA-SPON® STANDARD should be removed when bleeding is controlled. The technique currently used to ensure that GELITA-SPON® STANDARD is not aspirated into the patient's nose or mouth can only be used if the gel is dry and the product is removed from the nose or mouth.

By absorbing fluid GELITA-SPON® STANDARD may expand and impinge on neighbouring structures. Therefore care should be taken to avoid overpacking, particularly within bony cavities, as it may lead to interference with normal function and may eventually lead to compression necrosis of surrounding tissue.

In cases of postoperative infective operation may be necessary to remove infected material and allow drainage.

Cell save: The use of GELITA-SPON® STANDARD with a cell saver is not contraindicated. However, make sure to use proper technique when using GELITA-SPON® STANDARD in conjunction with a cell saver. As with other hemostatic agents, GELITA-SPON® STANDARD should not be aspirated into autogenous blood salvaging systems.

1. Prior to inflating the use of GELITA-SPON® STANDARD is not contraindicated. The technique currently used to ensure that GELITA-SPON® STANDARD is not aspirated into such devices is as follows:

2. Apply GELITA-SPON® STANDARD according to the IFU until hemostasis is satisfied.

3. Gently irrigate excess GELITA-SPON® STANDARD with your preferred irrigation solution, using the wall suture to remove all loosened matrix. All matrix not removed by prior irrigation/flushing is incorporated into the clot.

4. **General storage and handling:** GELITA-SPON® STANDARD should be stored dry in its sterile protective outer packaging at room temperature avoiding contact with direct sunlight.

Remove the product from the sterile package using sterile technique. Prior to opening, the integrity of the sterile barrier should be confirmed. Discard the packaging if damaged.

Any unused contents should be discarded.

The packaging is subject to and conforms to European & International legislation and other applicable standards. The packaging protects the product from external influences and guarantees sterility during storage.

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